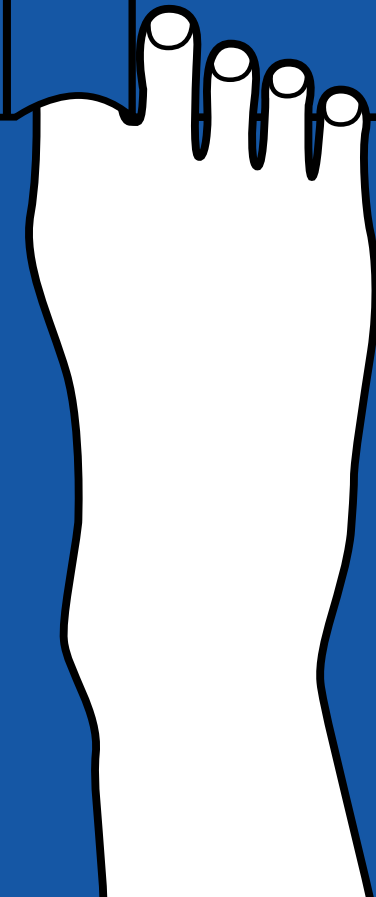
EASY

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Charge

Art.Nr.

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To easily and conveniently put on your stockings while sitting and without applying force.

By means of a tight fit support and compression stockings effect pressure on the legs. Corresponding to the human anatomy the closest fitting and highest pressure of the stockings is at the ankle. Therefore correct donning and doffing requires physical strength, skill, agility and a special technique.

COMPRESSANA EASY is made of an especially silky fabric. Once slipped around the foot and leg, it allows you to pull on stockings even over moist or lotioned skin, wound covers or stockings worn underneath.

COMPRESSANA EASY is equally suited for open as well as closed toes.

In addition usage of special gloves with a grip on the palm is favourable, as recommended in the user manual of the hosiery manufacturers. These are available as accessories.

COMPRESSANA EASY protects the skin against injuries and the fabric against damage caused by mechanical influences, e.g. friction, fingernails, toenails or jewellery, especially in combination with COMPRESSANA GRIP special gloves. In this way the stockings keep their therapeutic effect.

COMPRESSANA EASY makes the start into the compression therapy easier and ensures that the stockings are donned correctly.

COMPRESSANA EASY is suitable for people who prefer an easier way of donning or people in special life circumstances, e.g. limited mobility.

Donning aids can be reimbursable when available on prescription.

Donning and doffing aids are recommended or prescribed by the doctor if the donning of the stockings without therapeutic appliances is too difficult. For example in cases of illnesses or disabilities with considerable dysfunctions of the finger, hand and arm joints as well as for people with limited finger grip:

- high-grade inflammatory or substantially wearing off joint illnesses
- signs of paralysis
- deforming illnesses in the hand part
- consequences of injuries at the hand part
- congenital anomalies
- broad spinal, hip and/or knee joint stiffening
- chronic back pain
- adipositas permagna

Space-saving – ideal for travelling

TEST IT FOR FREE!

If you are not sure if **COMPRESSANA EASY** is a real alleviation you are welcome to test the donning and doffing aid without any engagement.

DETAILED INSTRUCTION!

Our partners in pharmacies or medical specialized trades will demonstrate how to use **COMPRESSANA EASY** and will be on hand with help and advice for the first instruction. In the beginning practice donning and doffing with **COMPRESSANA EASY** several times. After a short time you will gain routine in the handling. Please consider the following directions for donning and doffing as recommendations that offer the possibility to perfect the handling or to adapt it to your own style.

SPECIAL TIPS:

- Do not pull **COMPRESSANA EASY** out by the straps.
- Never pull the stockings at the top or at the top grip upwards.
- Upend the silicone beaded band before donning.
- Use **COMPRESSANA EASY** only when sitting.

In case of open wounds in the application area provide for a professional wound management and sore treatment. Do not use COMPRESSANA EASY directly on open wounds.

For hygienic reasons, this product is intended for one person only.

DONNING

Variation 1: COMPRESSANA EASY lies under the heel

Lay the donning aid COMPRESSANA EASY on the floor and place your foot on the cloth.

Slip one or several toes into the provided opening in the hem.


Loosely tie the cloth around your calf – yet not too high. Now wrap the cloth across your foot – first on the inside, then outside.


Variation 2: COMPRESSANA EASY lies across the instep

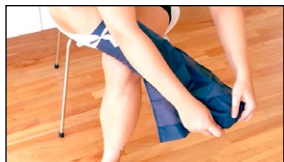Lay the donning aid COMPRESSANA EASY onto your foot and your shin.

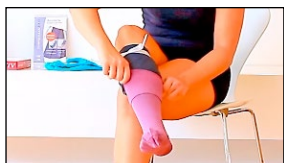Loosely tie the cloth above your calf – yet not too high. Now slip the provided opening in the hem over one or several toes.

Next with both variations:

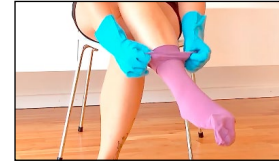
It helps to turn your stocking only halfway inside out. Slip the stocking over your foot and your donning aid.

Undo the straps of the donning aid and lift the hem of the cloth over your toe.

Grab the cloth of the donning aid with both hands and pull it out of the stocking - bit-by-bit like a rope. **Do not pull on the straps.** Pulling it out can be easier by slipping the stocking back a bit.

Grab the stocking by the cuff and slip it up. Wear COMPRESSANA GRIP, TOP-GRIP or SUPER-GRIP gloves when doing so.

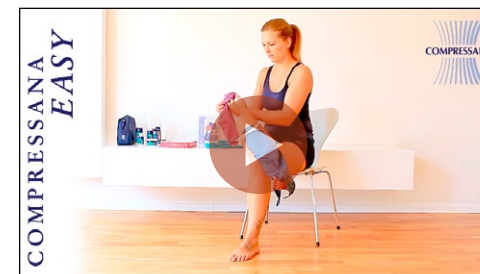
Evenly spread the knitted fabric with the gloves. Make sure that it lies flat and smooth, without any creases.

DOFFING

GRIP special gloves facilitate doffing of stockings and are available as accessories. Push the stocking down the leg by sliding the stocking step by step downwards with the help of the special surface texture of the gloves.

Do not leave COMPRESSANA EASY on the floor.
Considerable danger of slipping!

Watch COMPRESSANA EASY
in action
www.youtube.de/compressana

MATERIAL COMPOSITION:

100% Polyester

WASH AND TEXTILE CARE SYMBOLS:

Hand wash at low temperature (lukewarm, approx. 30° C). Use mild washing detergent (**COMPRESSANA CLEAN Art.No. 0121**).

Air-dry or wipe dry with a cloth.

Do not bleach, do not dry-clean or use any solvents.

Do not iron. Do not expose to heat (e.g. radiator).

DISPOSAL

There are no specific criteria for disposal. All components when worn-out or at the end of service life or lack of the ability to function can be disposed of with household waste.

DURABILITY / DURATION OF USE / STORAGE:

Keep away from UV radiation. Store in a dry place and at room temperature. Avoid direct sunlight, heat and moisture. The application time depends on the intensity of use and the type of fabric or thickness of the stockings. Regular use results in a material-related fair wear and tear.

Check COMPRESSANA EASY before use and application thoroughly for defects and proper functioning. In case of damage contact your specialist dealer or your pharmacy.

REORDER/ACCESSORIES

Art.No.	Size
COMPRESSANA EASY Donning-aid	
0098	OneSize
Clean detergent 250 ml	
0121	–
COMPRESSANA GRIP special gloves	
0102	S Small < 7,7 cm
0102	M Medium < 8,7 cm
0102	L Large < 9,8 cm

MEANING OF THE SIGNS ON THE PACKAGING OR THE PRODUCT

Manufacturer

Charge

Art.No.

Take note of the instructions for use.

Attention: please heed product advice

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COMPRESSANA GmbH