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english

COMPRESSANA CALYPSO offers the latest hosiery in many forms from knee socks to extra-wide tights. Choose between soft to forte support effect and shiny to opaque surface.

The combination of natural cotton and microfibres makes **COMPRESSANA Microcotton** so skin-friendly, breathable and ensures optimum comfort and an attractive appearance.

These stockings and tights should not be pulled on but put on with the palm, without tugging, if possible. Ensure that the knitted section is evenly distributed over your leg. Adjust the fit after putting on with COMPRESSANA stocking gloves. When putting them on, take care that your toes are not constricted. Pull the toe of the stocking slightly forward if necessary.

Care instructions

Your hosiery will last longer if you take good care of your feet. Rings and sharp fingernails can damage the fine mesh. The same applies to damaged shoes. Even after many washes **CALYPSO support hosiery** will keep their good look and effectiveness:

- **Hand wash** with mild detergent in warm water. Rinse well, do not wring!
- **Machine wash** use mild detergent and delicate cycle at up to 40°C We recommend using COMPRESSANA laundry bag.
- **Drying:** Wrap in a towel or hang up on the washing line. Do not dry on a heater or in the heat of the sun! Do not put in the dryer!

Wash before wearing • Wash inside out • Do not use any softener

Application / Donning and maintenance instructions

Please observe the information on the packaging, the enclosed product leaflet, or the instructions of your physician, pharmacist or medical specialist. In the event of difficulties in donning or using the product, you can find more in-depth instructions/information and accessory-related videos at www.compressana.de/downloads and [youtube/compressana](https://www.youtube.com/watch?v=...). Alternatively, please consult your medical specialist / pharmacist / specialist retailer. COMPRESSANA donning aids and special-purpose gloves can simplify donning and removing the stockings.

Durability / Application period / Storage

Store in a dry, dark place at room temperature to protect against UV radiation. Avoid direct sunlight, heat and moisture. The application period depends on the intensity of usage. Depending on the application and material, the stockings will suffer wear, tear and natural deterioration during regular use. Check the stockings thoroughly for flaws and fitness for purpose before use. In the event of any damage, consult your specialist retailer or pharmacist..

Risks / Side effects

When used incorrectly or worn in the wrong size or fit, support stockings can cause skin necrosis, oedema or pressure damage to peripheral nerves. If you notice any pain or negative changes (e.g. skin irritation, constrictions) during use, remove the stockings and **consult your doctor or pharmacist / medical specialist without delay**. In the event of somatosensory disorders and/or wound-healing disorders (e.g. diabetic illnesses) in combination with skin lesions, poorly-healing wounds can occur unnoticed. People in this category require special monitoring and specific instructions regarding specific risks and preventive methods.

Do not wear the stockings in the event of open wounds or suppurating dermatosis/skin diseases. Alternatively, ensure that any wounds are properly dressed before donning the stockings.

The manufacturer accepts no liability for any damage or injury due to improper use, non-adherence to the instructions or as the result of using accessory products not manufactured by COMPRESSANA. Inform your doctor if the effect is insufficient to ease any existing complaints.

Disposal

There are no special criteria for disposal. The stockings can be disposed of via the household waste, the packaging via the paper recycling.

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